

NON-ALCOHOL SKIN BARRIER LIQUID REMOIS COAT




Moisturizing barrier protects the skin from irritation and staining.

REMOIS COAT Skin-friendly skin barrier.

Protecting and **Moisturizing** have effect of maintaining a healthy skin.

Uncomfortable sealed-in feeling

Microparticle structure with moisture permeability and waterproof can prevent skin tightness and sealed-in feeling.

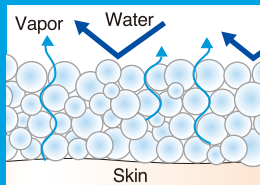

Worries about irritation to the skin

Non-alcohol preparations guard better against irritation. Fast-drying, without leaving a sticky feeling. You can apply adhesives, such as tape, after applying it*. The added moisturizer will keep the skin moist.

*You should confirm adhesion after application.

Worries about getting the hands dirty

The spray type can be applied with one hand, and can be used for large areas without touching. The clear bottle makes it easy to see how much is left.



Incontinence Area

The skin barrier formed by microparticles can protect the skin from irritation and staining.

Adhesive Application Area


Peristomal Skin

A skin barrier can protect the skin from irritation and staining.

Instruction for REMOIS COAT


*Read the Instruction manual before you use this product.

REMOIS COAT

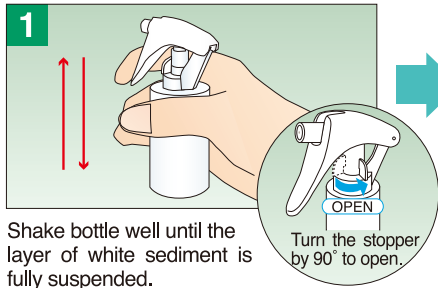


Shake bottle well until the layer of white sediment is fully suspended.

Turn the stopper by 90° to open.

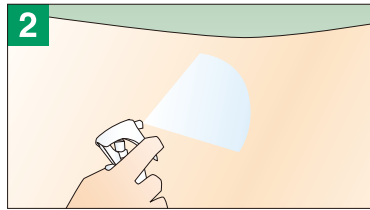


Spray REMOIS COAT directly and evenly onto the relevant area.

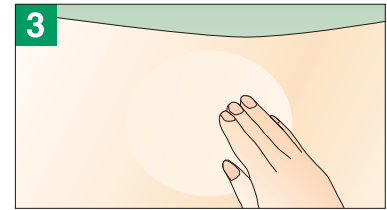


Let the skin dry naturally: Do not rub with the fingers or gauze, etc. Touch the skin lightly. The skin barrier is complete if it is not sticky.


REMOIS COAT HANDY

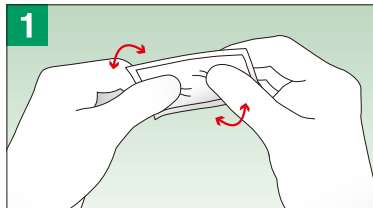

Before opening an aluminum pack, knead it between your fingertips four or five times. This will loosen its surface and make it easier to pull out the REMOIS sheet.

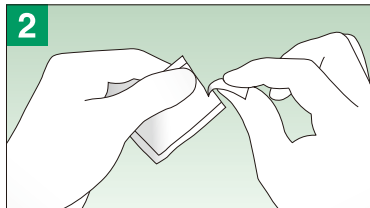

Open the aluminum pack and pull out the sheet, making sure to keep the opening upright to prevent any liquid from spilling out.

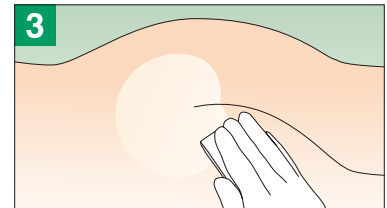

Apply the sheet evenly to the relevant area. Let the area dry naturally beforehand: Do not rub with the fingers or gauze, etc.

* Do not apply to mucous membranes or other sensitive areas. If adhesives such as tape are used, apply them only after the skin has completely dried.

Precautions

- Do not apply to broken skin or skin that has eczema or dermatitis.
- If abnormalities, reddening, irritation, etc., are noticed during or after application, immediately stop using and contact a medical specialist. These symptoms may become worse if it is continually used.
- If the spray falls on mucous membranes such as the eyes, etc., immediately wash it off with cold or lukewarm water.
- Be careful not to inhale the product through the nose or mouth.
- Apply the product in a well-ventilated place. It is **flammable**, so keep it away from fire.
- Keep away from infants and children.
- Stop using the product if you notice any skin problems.



REMOIS COAT [Cosmetic liquid]

Type	Product Code No.	Specification	Contents Per Box
—	17711	30ml	1 bottle

REMOIS COAT HANDY [Cosmetic liquid]


Type	Product Code No.	Specification	Contents Per Box
—	17712	1.7ml	20 packs



What's REMOIS?

ALCARE proposes a new concept for skin care, "Skin Care" + "Moisture"

Moisture is added on procedure required for skin care and through the improvement of the barrier function, it will keep the skin condition healthy.

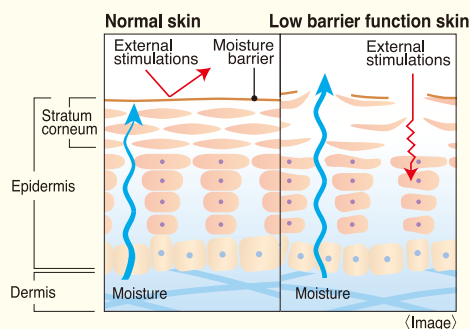

The skin with the lower barrier function will dry out due to the damaged stratum corneum, and so it will be easy to be exposed to the external stimulations by allergen* or microorganisms. To keep the skin condition healthy, moisturizing the stratum corneum is significant.

*Allergen: It causes allergy reaction. There are many allergen, dead mites, formaldehyde, etc.



ALCARE Co., Ltd.

International Sales Unit
19F, Arcacentral Bldg., 1-2-1 Kinshi, Sumida-ku, Tokyo 130-0013, Japan
TEL. +81-3-5611-7819 FAX. +81-3-5611-7827 MAIL. info@alcare.co.jp
www.alcare.co.jp/e